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## Produktidentifikation

Produktname: Hybrid Recon

Marketingname: Hermia Reconstruction

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The screenshot shows the 'Program parameters' dialog box with the 'Recon' tab selected. The dialog is organized into several sections:

- Cardiology protocols:** Includes fields for 'Study 1', 'Study 2', and 'Study 3'. Below these are 'Primary non-gated', 'Primary gated', and 'Secondary non-gated' fields. A 'Secondary NG' radio button is selected. A 'Count limit [%]' dropdown is set to '20'. 'Outline' and 'Extend transverse FOV' radio buttons are present.
- Lung protocols:** Includes 'Study 1' and 'Study 2' fields.
- Neurology protocols:** Includes 'Primary', 'Secondary', and 'Tertiary' fields. The 'Tertiary' field contains 'hermes\_rec\_default'. Radio buttons for 'Perform secondary reconstruction' and 'Perform tertiary reconstruction' are present.
- Oncology protocols:** Includes 'Primary', 'Secondary', and 'Tertiary' fields. The 'Primary' field contains 'onco\_rec\_default', 'Secondary' contains 'onco\_nac\_rec\_default', and 'Tertiary' contains 'hermes\_rec\_default'. Radio buttons for 'Perform secondary reconstruction', 'Perform tertiary reconstruction', and 'Reconstruct whole FOV in NAC' are present.
- Mumap display zoom [%]:** Set to '100'. 'Uniform mumap display zoom [%]:' is set to '265'.
- Warning options:** A red box highlights the 'Skip frame of reference warning' radio button, which is selected. Other options include 'Skip subset warning', 'Skip energy window warning', and 'Skip mumap truncation warning'.
- Reduce number of reconstruction methods:** A radio button option.
- Mumap transformation:** Radio buttons for 'Automatic full 6 parameter', 'Automatic translation only' (selected), and 'Manual'.
- Mumap page screen cap label:** An empty text field.
- Save:** A button at the bottom.

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**Hermes Medical Solutions Ltd**
**Hermes Medical Solutions, Inc**
**Hermes Medical Solutions Canada, Inc**
**Hermes Medical Solutions Germany GmbH**